COLPHARMA  [®]

Air 2000

P O C K E T

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Read the instructions carefully before using this device.

BF type part applied.

Dear Customer,

Air 2000 Pocket by Colpharma is a next-generation micro mesh nebuliser for aerosol therapy. It is completely silent, light weight, compact and portable, with battery or USB cable power, and it has been designed to treat respiratory diseases with aerosol therapy. It can be used with the most common aerosol therapy medicines. If you have any questions or problems or if you need to order spare parts, please contact your dealer or the COLPHARMA customer care service, or visit www.colpharma.com. The mesh is in metal, with characteristic microscopic holes. This membrane is vibrated at high frequency, nebulising the medicine into ultra-fine particles for effective aerosol therapy.

INTENDED USE

This product has been designed for the inhalation of medicines for the treatment of respiratory illnesses.

Intended audience

- Certified medical professionals (physicians, nurses and therapists).
- Patients at home with home healthcare support for treatment under the directions of a qualified physician.

The user must understand the general operation of the device and the contents of the instruction manual.

It is forbidden to use the device on patients who are unconscious or who cannot breathe on their own.

Product recommended for:

people who suffer from respiratory problems and who need to take medicine by inhalation. Suitable both for paediatric and adult use.

Environment

This product has been designed for home use, as well as for medical clinic settings and hospital use.

Minimum lifespan of the device

The minimum lifespans of the device are shown below and are based on the assumption that the product is used to nebulise a saline solution (2 ml) twice a day, for 7.5 minutes at a time, at an ambient temperature of 23 °C.

The duration may vary according to the environment of use and to the correct cleaning of the micro mesh nebuliser.

- Main unit: 3 years.

- Nebuliser unit, mouthpiece, paediatric mask, adult mask: 1 year.

Frequent product use may shorten the expected lifespan.

Warnings for use

It is necessary to follow the warnings and precautions provided in the instruction manual and to scrupulously follow the cleaning and disinfection instructions for the micro mesh nebuliser.

IMPORTANT PRECAUTIONS

- Follow the instructions for use. This manual contains important information on the operation and safety of this device. Please read the instructions carefully before using the device, and keep them handy for future reference.
- Like all medical devices, this product can become unusable in the event of a power outage, if the batteries should become flat, or in the event of a mechanical problem. It is recommended to have spare batteries available and a backup device if it is not possible to postpone treatment.
- When using electrical products, it is necessary always to follow the basic safety precautions. Like for all electrical devices, take particular care with children.
- Use this device only with medicines prescribed by a physician and follow the dosing, duration and frequency instructions for the prescribed treatment.
- This nebuliser is exclusively for use in aerosol therapy; any other use of the device is improper and therefore, dangerous. The manufacturer cannot be held responsible for any other damage caused by improper or incorrect use.
- The device is not suitable for anaesthesia or pulmonary ventilation.
- The device must be used only with its original accessories, as listed in this user manual.
- Do not share the nebuliser with other people. This product (mask and mouthpiece included) must be used by a single patient.
- Before use, make sure that the device and all its parts and components (for example, mouthpiece and mask) are correctly assembled, as described in these instructions for use.
- Clean and disinfect the nebuliser and accessories (mouthpiece and mask) as described in the section "Cleaning and disinfection". Make sure that the nebuliser and accessories have been cleaned and disinfected correctly before use, to avoid all possible contamination.
- The patient is the intended operator of the nebuliser. Adult supervision is necessary if the device is being used by a child or person requiring special assistance.
- This device is for human use only.
- For safety reasons, always disconnect the USB cable from the USB port in the following situation:
 - if the power adaptor is damaged
 - if a malfunction has occurred during operation
 - before cleaning the device
 - immediately after use.
- Do not use the device if you deem it damaged or if you notice any anomaly.
- Never try to open, repair or modify this device. Do not try to clean the micro mesh nebuliser with foreign objects, as there is a risk of damage.
- This device has been manufactured using sensitive components and it must be handled with care. Follow the storage and operation instructions in the "Technical Specifications" chapter.
- Never use this device when the ambient temperature is above 40 °C. For further requirements regarding the operating environment, see the "Technical Specifications" chapter.
- Make sure to safeguard the product from:

- water and humidity
- extreme temperature
- impacts and falls
- contamination and dust
- direct sunlight
- heat and cold

- Store the device and its accessories in a clean place.
- Do not shake the nebuliser during operation.
- Do not use or store this device if smoke or gas is present.
- Do not use any medicines left in the nebuliser. Always use a fresh new dose.
- Never handle the power adapter with wet hands.
- Never use the device if the USB cord or power adaptor are wet.
- Never remove the power adaptor from the main socket by pulling the cord.
- Do not swallow small parts of the nebuliser.
- Never use water to wash the power unit, adaptor or USB cable. If any liquid is spilled on the power unit or adaptor, clean them immediately.
- Never place the power unit in water or other liquids, and be sure to protect them from splashes or sprays of water or other liquids.
- Do not use the device in damp environments such as bathrooms.
- Do not use the device close to strong magnetic fields such as cellular phones and radio installations. Keep a minimum distance of 3.3 m from other equipment when using this device.
- When connecting the power unit, make sure that the voltage at the power socket is correct and make sure that the plug does not cause an overload.
- Disconnect the power after using the device, and never leave a device unattended if under tension.

Children must always use this device under adult supervision.

Some parts are small and may represent a choking hazard if swallowed. Cables and tubes may represent a choking hazard.

Electronic devices must be disposed of according to local regulations and not as ordinary household waste.

The use of this device does not replace the medical advice of a physician.

HOW TO PREPARE AND USE THE DEVICE

When using the device for the first time, we recommend cleaning all parts as described in the "Cleaning and Disinfection" section.

POWER SUPPLY

Operating LED status	Status meaning
Green lit	Device operating
Orange lit	The battery is flat
Green flashing	The device is entering self-cleaning mode

INSERTING THE BATTERIES

If using batteries to power the device, open the lid to the battery compartment (bottom of the power unit) following the direction of the arrows (Pic. A). Fit two AA batteries into the battery compartment, according to the signs (+) and (-) on the compartment lid to be sure of inserting the batteries in the right direction.

Close the lid. When the operating LED turns orange, replace both batteries with new ones. The duration of the batteries depends on their capacity and their condition.

NEVER insert a battery with the positive pole (+) and negative pole (-) facing the wrong way.

NEVER use old and new batteries together and do not mix batteries of different brands.

USING THE USB CABLE

If using the USB cable, connect the micro USB to the micro USB port in the back of the power unit (Pic. B). Connect the USB cable end to the power unit. The unit must be able to provide a direct current of 5V/1A and may be a standard charger or power bank.

DEVICE ASSEMBLY

1. Fitting the nebuliser unit. Push the nebuliser unit into the slot in the power unit, following the direction shown by the arrow (Pic. C). A "click" will indicate that the nebuliser is in position. If the nebuliser unit is not correctly installed, the nebulisation might not be effective.
2. Remove the nebuliser unit from the power unit. Hold down the PRESS button on the power unit and take the nebuliser unit out of the power unit.

Caution: to prevent damage to the device, never take out the nebuliser unit without first having pushed the PRESS button.

USING THE DEVICE

1. Open the medication container, holding the power unit in one hand and gripping the tab on the lid between the thumb and forefinger of the other hand to release it by pulling upwards (Pic. D):
2. Add the amount of medicine prescribed by the physician (Pic. E). The maximum and minimum volume of the fluid is 8 ml and 0.5 ml respectively. Never exceed the LIMIT (max. 8 ml).
3. Close the medicine container lid: a "click" sound will indicate that it is correctly closed (Pic. F).

4. Fitting the mouthpiece: connect the mouthpiece to the connection of the nebuliser unit (Pic. G) To use the mouthpiece, close your mouth gently over it so as not to dispel the flow of the medicine being nebulised (Pic. H).
5. Fitting the mask: connect the mask to the connection of the nebuliser (Pic. I). Bring the mask up to the face (Pic. J) and adjust the elastics to secure it comfortably.
6. Operation: press the ON/OFF button to turn on the device. The operation LED will light up green and the device will start to nebulise.
7. Inhaling: keep the device in one hand, slowly inhaling the nebulised medicine by breathing in through the nose or mouth. Breathing slowly and constantly boosts the inhalation of the aerosol medicine into the respiratory tract.

Warning:

- Do not ingest aerosol medicines.
- Do not shake the device firmly during use to avoid anomalies during operation, such as automatic power off.
- The duration of each inhalation session should not exceed 15-20 minutes.
- For aerosol treatment to be effective, it is necessary to be calm, relaxed and seated. An uncomfortable posture and non-constant breathing may cause obstruction in the airways.
- Use the mouthpiece correctly to prevent a build-up of medicine on the tongue.
- Press the ON/OFF button to interrupt the aerosol therapy, when required. The device powers off automatically when the medicine has finished.
- When there are lots of bubbles between the medicine and the micro mesh nebuliser, the device will not be able to nebulise the medicine. It is possible first to switch off the device then to shake the nebuliser gently. Then restart the device (Pic. K).
- To guarantee the normal nebulisation of the medicine, keep the device vertical or tilt it slightly towards the micro mesh nebuliser. If the operational device is tilted to the point that the liquid is not in contact with the micro mesh nebuliser, the device will switch off after 10 seconds.
- Slightly tilt the nebuliser towards you when the solution is almost finished to make sure that any remainder stays in contact with the micro mesh nebuliser to be nebulised completely.
- A high-viscose solution may not nebulise fully or may even clog the micro mesh nebuliser. In this case, switch off the device and use distilled water to rinse out any built up solution.

ENDING TREATMENT

1. At the end of the treatment, press the ON/OFF button to switch off the device. Remove the mouthpiece or mask from the nebuliser unit.
2. Pour away any residual medicine and clean the device. For more information, see the "Cleaning and Disinfection" section.

CLEANING AND DISINFECTION

Cleaning

- The nebuliser unit and accessories (mouthpiece or mask) must be cleaned and disinfected after every use.
- After every use, remove the mouthpiece or mask from the nebuliser unit.
- Open the top lid on the nebuliser unit to eliminate any residual medicine.
Add tap water to the medication container and gently shake it, then pour out the water. Take care not to wet the outer surface of the device.
- Add more tap water and close the nebuliser. Press and hold down the ON/OFF button for 5 seconds; the LED will flash green and the device will enter the self-clean mode. After cleaning for 2-3 minutes, press the ON/OFF button to interrupt the cleaning mode. Then pour out the remaining water.
- Remove the nebuliser unit from the power unit. Disconnect the USB cable from the power unit if using the USB power supply. Wash and rinse the nebuliser unit with distilled water (the power unit must not be washed). Shake out the excess water and leave to dry naturally on a clean, dry towel. Do not touch the micro mesh nebuliser with fingers, cotton buds or other items.
- If necessary, use a piece of gauze or a clean towel to remove any marks from the power unit.
- Make sure that all clean parts are completely dry before putting them away or using them.
- The power unit contains electronic parts and electrodes. Do not immerse or rinse the power unit in water. There is a risk of electric shock or loss of current, with resulting malfunction of the device.
- Do not wash device parts in the dishwasher.
- Never use a microwave to dry parts.
- When the micro mesh nebuliser is seriously clogged, it cannot be cleaned with the above methods; it is necessary to try the following method: add boiling water and a few drops of white vinegar to the medicine container and activate the cleaning mode. If the micro mesh nebuliser is still clogged, replace the nebuliser unit.
- Clean the mask and mouthpiece with a mild detergent before using for the first time and after every use. Hand wash the mask and mouthpiece in hot water (30°C) with a mild detergent, rubbing gently for at least 30 seconds. Rinse thoroughly in distilled water.
Leave them to dry naturally, well away from direct sunlight.
- Make sure that the nebuliser unit and accessories are completely dry and that there are no liquid residues in the nebuliser before storing it.

Disinfect with alcohol

- Rinse the nebuliser unit and accessories in distilled water.
- Immerse the nebuliser unit and attachments in 75% alcohol (ethanol) for 10 minutes.
- Rinse the nebuliser unit and the accessories again, using distilled water. Shake out any excess water and leave to dry naturally on a clean, dry towel.
- Make sure that all parts are clean and dry before storage or use.

- Alcohol is highly flammable. Never use alcohol near naked flames or smoke.
- Do not rinse or soak the power unit, power adaptor or USB cable in any liquid.
- Before storing the device, make sure that the nebuliser unit and accessories are completely dry and that there are no liquid residues in the unit.

MAINTENANCE, STORAGE AND CARE

Order all the spare parts from your retailer and trusted pharmacy or contact the COLPHARMA customer service.

- Make sure there are no liquid residues in the nebuliser unit.
- Make sure that the nebuliser unit is cleaned and disinfected carefully after every use.
- Store the device and accessories in a clean, safe place.
- Remove the batteries if the device will not be used for a prolonged period of time. Failure to do so will risk damage due to battery leaks.
- Remove the batteries if you intend to take the device with you when out of the house or travelling.
- Protect the device from direct sunlight or impact.
- Do not place the device in water, under running water or in very damp environments.
- Keep the device well away from heating or naked flames.
- Protect the device from contact with corrosive fluids.
- Never use a microwave oven to dry the device or its accessories.
- Never wind the power cord around the device.
- Always use or store the device in an environment where the temperature and moisture levels are compliant with the specific requirements in the "Technical Specifications", otherwise the device may not operate correctly.
- If the device does not work, see the solutions in the "Troubleshooting" section.
- Never use the device in places where it could be exposed to flammable gases.
- This device conforms to the EMC IEC60601-1-2 standard. If the device is used together with other medical devices or electrical equipment, they may influence the operation of the one of the devices. Never use the device together with other medical devices or electrical equipment.

Warning:

Before the device is returned to the manufacturer for inspection and repair, do not disassemble it or attempt to repair it without authorisation.

TROUBLESHOOTING

If failures persist after trying any of the measures above, do not try to repair the device; contact the COLPHARMA customer sales service.

N.	Problem	Possible causes	Possible causes
1	Nebulisation is weak or non-existent	The battery flat indicator switches on / flat battery.	Replace the batteries and restart the device.
		The holes in the micro mesh nebuliser are clogged.	Clean the nebuliser unit (check the "Cleaning and Disinfection" section). Replace if necessary.
		The medicine is finished or no longer in contact with the micro mesh nebuliser for more than 10 seconds.	Tilt the device towards the user to allow the medicine to come into contact with the micro mesh nebuliser.
		The micro mesh nebuliser is damaged.	Replace the nebuliser unit.
2	The operating LED does not switch on and the device is not working	The batteries are inserted facing the wrong way.	Follow the (+) and (-) on the lid of the battery compartment to refit the batteries in the correct direction.
		Low battery voltage.	Replace the batteries or use the USB cable to power the device.
		The USB cable is not correctly connected to the power unit or power.	Reconnect the USB cable correctly to the power unit and power supply.
		Lack of power or USB cable damaged.	Replace the power supply or USB cable.
3	The operating LED switches on but the device is not working	Incorrect nebuliser unit installation.	Refit the nebuliser unit correctly.
		The battery flat indicator switches on / flat battery.	Replace the batteries and restart the device.
		The power contacts on the power unit or those in the nebuliser unit are dirty.	Clean the power contacts with a medical gauze.
		The angle of the device is too great and the medicine cannot come into contact with the micro mesh nebuliser.	Reduce the tilt angle of the device or tilt it towards the user to allow the medicine to come into contact with the micro mesh nebuliser.

4	The device switches off automatically during use	The angle of the device is too great and the medicine cannot come into contact with the micro mesh nebuliser.	Reduce the tilt angle of the device or tilt it towards the user to allow the medicine to come into contact with the micro mesh nebuliser.
		The device shakes a great deal during use.	Hold the device gently during use.

WARRANTY

This device has a **3-year** warranty from the date of purchase. The warranty is valid only with the original purchase receipt

- The warranty covers the device against defects in materials and workmanship. The nebuliser unit and micro mesh nebuliser are not covered by the warranty if the malfunction (clogging of the micro holes) is caused by bad maintenance or unsuitable medicines.
- The spare parts such as nebuliser, masks, mouthpiece, and air tube are guaranteed for one year against defects in materials and workmanship, but not against deterioration caused by normal wear and tear.
- Opening or tampering with the device voids the warranty.
- The warranty does not cover damage caused by improper use, accidents or failure to abide by the instructions for use.

CLASSIFICATION

- Appliance not suitable for use in the presence of flammable mixes.
- BF type part applied. The mask or mouthpiece are recognised as an applied part.
- Appliance powered internally (without adaptor), class II equipment (with adaptor).

TECHNICAL SPECIFICATIONS


Power supply:	2 AA batteries or adaptor (AC-DC 5 V), 1 A)
Rated power:	<3 W
Ultrasound frequency:	Approx. 110 kHz
Safety level:	Type BF, Class II
Nebulisation speed:	~0,30 ml/min
Maximum filling volume:	8 ml
Minimum filling volume:	0,5 ml
MMAD:	~3,5 µm
FPD:	~80%
Flat battery indicator:	1,9 V ± 0,2 V
Useful life:	
- Power unit:	3 years
- Nebuliser unit	180 hours
- Accessories:	1 year
Automatic power off:	Automatic power off in absence of liquid
Operating temperature and humidity:	+10 °C ~ +40 °C, 30%~85% R.H.
Storage temperature and humidity:	-10 °C ~ +40 °C, 10%~85% R.H.
Altitude:	≤2000 M
IP class:	IP22
Size:	98 mm × 45 mm × 73 mm
Self-cleaning mode:	Yes
Reference standard:	EN 13544-1+A1:2009, EN 62304, IEC 61000-4-2:2008, IEC 61000-4-4:2012, IEC 61000-4-5:2005, IEC 61000-4-11:2004, IEC 61000-4-8:2009, IEC 61000-4-6:2013, IEC 61000-4-3:2006+A1:2007+A2:2010, IEC 60601- 1-2:2014, IEC 61000-3-2, IEC 61000-3-3

The technical specifications can undergo variations without warning. Performances may vary with medicines such as suspensions or high viscosity. For more details, see the technical data sheets for the medicine suppliers.

Particle dimensions diagram

According to the regulation EN 13544-1:2007+A1:2009 "Respiratory therapy equipment - Part 1: nebulising systems and their components", annex CC.3.

Test conditions	
Test temperature	24+-2C°
Test humidity	45%-75%
Test solution	NaF 2,5% (M/V)

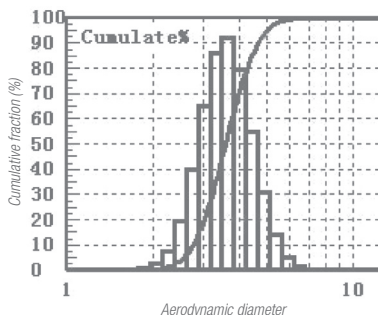REGULAR SAFETY CHECKS

If using a device with the power adaptor, the preventive inspection and maintenance to be performed, including the frequency of said maintenance are:

- Check the adaptor before every use; if damaged, do not use.
- Clean the adaptor plug at least once a year. Excess dust on the plug can be the cause of fire.

In the interest of progress, the manufacturer reserves the right to make technical changes without prior warning. In case of changes to this manual, no prior warning will be given. The trademarks and names mentioned are the property of their respective owners.

DISPOSAL

This product must not be disposed of with other household waste. Users must dispose of the equipment for waste by taking it to a waste recycling centre for electrical and electronic appliances. For more information about these collection points, contact your local authority, your local waste disposal service or the store from which you purchased the product. Incorrect disposal may be subject to fines, according to national laws.

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HONSUN (NANTONG) Co., Ltd
Shanghai International Holding Corp. GmbH (Europe)
MedNet SWISS GmbH,